The start screen contains the latest news about the software.

The expiry date of the software licence is also shown in green in the upper left-hand part of the screen.

The "START (RECOMMENDED)" button takes you to the material settings.

With "START (PREVIOUS SETTINGS)" you skip these and can import the scan data directly.

ASM	Welcome			2₀ ≡
	Welcome to SAM A	<section-header><section-header><section-header><section-header><list-item><list-item><list-item><section-header><section-header></section-header></section-header></list-item></list-item></list-item></section-header></section-header></section-header></section-header>	TTART (PREVIOUS SETTINGS)	START (RECOMMENDED)
		QUICK HELP REVIEW YOUR LICENSE AND THE NEWS. THEN START BY CLICKING	S'START'	

The setting options are used to define articulators as well as available materials.

If a certain distance block or mounting plate height is not directly available, you can deselect it here by clicking on the yellow tick next to the designation.

This block or mounting plate will then not be used for the calculation of the models.

With the button "NEXT IMPORT FILES" you only get to the file selection.

ASM	Settings			20	Ξ
	Make your ASM settings h	ere. Blocks at hand M Wee M We	Mounting plates in stock         ✓ Alf for sizes         ✓ 0.0 mm - ASM 120         ✓ S.0 mm - ASM 121         ✓ 3.0 mm - ASM 122         ✓ 7.5 mm - ASM 123	5	
		DUICK HELP ADAPT YOUR SETTINGS IF NECESS	ARY		



📟 SAM AxioS	SnapMount 1.4.5					– 🗆 ×
ASM	Import case files					\$≎ ≡
	Add your <mark>ST</mark>	-	IMPORT YOUR FILES NOW			
		Maxilla	8	I	Mandible	8
• • • • • • • •						
٢						
	GO BACK					
			<b>QUICK HELP</b> ADD YOUR			

## Here you insert the aligned scan data by clicking on "IMPORT YOUR FILES NOW".

The files should have been anatomically aligned beforehand and saved in the SAM coordinate system. This can be recognised by the curtain "AXP-aligned\_sam\_" (origin AxioPrisa Software) in front of the original file name.

Select mesh files X											
← → • ↑	← → ▼ ↑ 🖡 > Dieser PC > Desktop > ScanBib > Zahmodel Viade > Matches > SAM v 🕑 🔎 SAM durchsuchen							4	• =		
Organisieren - Neuer Ordner											
Dokumente	e		Name		Änderur	igsdatum	Тур			NUDART VALID FILES NOW	1
📰 Bilder			ASM sam axis-orbital-plane.stl		17.10.20	22 14:31		3D model file		IMPORT YOUR FILES NOW	<u>.</u>
📃 exocad			AXP-aligned sam axis-orbital-plane.2 -planes.stl		18.10.20	22 08:32		.3D model file			
📮 Schnittproj	jekte		AXP-aligned_sam_axis-orbital-plane.3axis.stl		18.10.20	22 08:32		. 3D model file			
🔚 Videos			AXP-aligned_sam_axis-orbital-plane.stl		28.03.20			3D model file			
Zahmodel	Viade		AXP-aligned_sam_Viade models by Trios AXP-Position.stl					3D model file	Mandible		
			AXP-aligned_sam_Viade models by Trios LowerJawScan.stl						Manuble	<u> </u>	
Chebrive - P	rersonal		AXP-aligned_sam_Viade models by Trios UpperJawScan.stl		18.10.20	22 08:32	STL	3D model file			
🤳 Dieser PC											
3D-Objekte	e										
🔚 Bilder											
Resktop											
			9					, 			
	Dateiname: A	XP-aligned	f_sam_Viade models by Trios LowerJawScan.stl" "AXP-aligned_sam	_Viade models by Trios UpperJaw!	Scan 🕤	Mesh files (*.s	stl *.ply *.	.obj) 🗸			
						Öffnen		Abbrechen			
-											
O											
	601	ACK									
				QUICK HELP ADD YOU							



Once the files have been selected, a moving preview of the two jaw scans is displayed. The assignment is made by the software. However, if this is not done correctly, it is possible to change by pressing the yellow "<- ->" button. The "NEXT CHOOSE ALIGNMENT" button takes you to the next step.



The origin of the data is queried here. If the alignment was done via AxioPrisa, this is entered here. If the origin is ModJaw, select the second option.





Here you define the cutting planes to remove excess artefacts and to give the model the base boundary. With "DISTANCE (mm)" you raise or lower the plane, with "ANGEL (°)" you tilt it. CUT MODEL" converts the selection.

ASM	Cut model	& ≡
© © © © ©	<image/>	Define distance and angle of the cut planes DISTANCE (mm) ANGLE (*) 1000000000000000000000000000000000000
	DUICK HELP ROUGHLY CUT YOUR MODEL IN RELATION TO THE BITE PLANE	

After cropping the scans, "NEXT FINE TRIM MODEL" takes you to the next step.

ASM	Cut model	20 =
	<image/>	Define distance and angle of the cut planes   DISTANCE (mm)   ANGLE (*)   - 100 + - 05 +
	QUICK HELP ROUGHLY CUT YOUR MODE	



Here the software rounds the model. The degree of rounding is determined for the slider at the top right. Areas that are removed are marked orange. The selection is implemented by "PERFORM SMOOTING".



When the orange areas are removed, the scan is optimised for printing. Complete the step with "NEXT CUSTOMIZE ASSEMBLY".





The AxioSnapMount software calculates the optimum fitting height of the models so that you can print in a material-saving way. In the preview window (white background) the selected distance blocks and mounting plates are displayed in the top right and bottom right. If certain distance blocks or mounting plates are not available, you can still exchange them here with the arrow buttons on the right.



Likewise, warnings are displayed in the window on the right if the pressure angle of the base would become too steep or if printing with the selected spacer blocks or mounting plates might be difficult or impossible. Use the arrow keys in the light grey window to vary the selection.

NEXT GENERATE MODEL" takes you to the next step.





A patient ID or patient name is required here and entered at the top right. This will later be stamped into the generated model and is used for later allocation of the models.

Below this, different filling strategies can be specified; there are three to choose from.

Am SAM AxioSnap	Mount 14.5	- 0	×
ASM	Review & generate final models	20	≡
	<image/>		
	QUICK HELP CUSTOMIZE MODEL GENERATION		

In addition to the solid filling strategy (Full model), honeycomb models (Honeycomb model) and models supported by stability struts (Light model) are also available.





Different information is stamped into the generated body. For example, the type of articulator, the distance block and the required mounting plate are written on the front. The date of creation is shown on the right side and the patient ID on the back. The "NEXT EXPORT" button takes you to the last step.



Here the export is offered in three different coordinate systems. To select, click on the respective surface and save the data locally on the hard disk.

SAM AxioSr	iapMount 1.4.5					- 0	×
ASM	Export models					20	≡
	Export your <b>ASM m</b>	nodels here.					
	Save case as files for SAM apps		Save case as files for 3shape Dental Designer		Save case as files for exocad		
•	-5-A-N	1-	sshape⊵		exocad		
•							
© 0							
0			tore				
	GO BACK				EXIT ASM APPLICATION S	TART A NEW CASE	
			EXPORT YOUR GENERATED MODELS TO EXTERI	IAL SOFTWARE OR SUP	PLIERS		

